
OPERATION OF THE SOFTCYCLE UNIT

Softcycle Unit is easy to operate using the keys shown in the following diagram.



WHAT DOES EACH KEY DO

ON/OFF/LOCK

This button allows switching the unit ON, OFF and LOCK. Move this key to the ON, the LCD display will light up and you will hear a beep.

Move this key to LOCK, the unit is locked and symbol is displayed on the screen.

Move this key to OFF, the unit is off.

PROG

The Softcycle has specifically 4 selected programs including STRESS, MIXED, TONE, URGE.

To change the program, simply press the PROG key. The screen will show the program selected. The next time you turn the device on, it will automatically display the last program you used.

When FIRST switched on the unit automatically goes to program URGE.

TIME

The Softcycle counts down the minutes of your session. This countdown is displayed on the screen.

In MIXED and URGE programs, you can change the length of your program by repeatedly pressing and releasing the TIME key. This allows you to select a program length of 10, 20, 30, 45, 60 or 90 minutes or continuous time. The length of time that you have selected will be displayed on the screen.

▲ Strength UP Button

Press this button to increase the strength.

▼ Strength DOWN Button

Press this button to decrease the strength.

SPECIFICATIONS

Model: Softcycle

Output: Setting 0-120 in steps of 0.5mA.

Channels: Single

Waveform: Asymmetrical rectangular

Mode: 4 modes

Adjustable Timer (only for MIXED and TONE program):

10, 20, 30 45, 60 or 90 mins or continuous time

Environmental Specifications Operating: Humidity: 20 to 93% RH

Temperature range: 10 to 35C

Atmospheric Pressure: 700hPa to 1060hPa

Storage: Humidity: Up to 93% RH non condensing

Temperature range: -20 to 70C


Atmospheric Pressure: 700hPa to 1060hPa

Waveform Information

There is 1 types of waveforms:

(1). Asymmetrical Bi-Phasic rectangular waveform

Amplitude

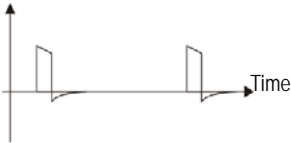

OTHER FEATURES

1. When the unit is turned on, if any of the keys are not pressed within 5 minutes, it will automatically shut off.
2. When the unit is turned on, press the TIME key and the UP key at the same time and hold 3 seconds, it will show the total time and average intensity. Press the TIME key and the DOWN key at the same time and hold 3 seconds, it will clear the total time and average intensity.
3. When the treatment timer has been set, it begins to count down one minute by one minute and is displayed on the LCD. Once it counts down to zero, the device automatically shuts off.
4. When the program changes, the output level will go down to zero immediately.
5. When the battery is low, the battery icon will flash indicating that the batteries should be recharged.
6. When the probe is not connected and the strength is increased to 10MA, you will hear 3 beeps. The output level will reset to zero and the yellow light and output indicator will flash.